# Phantom QA Results: 32 CH

#### Date:01/17/2017

#### **Dartmouth Brain Imaging Center (DBIC)**

Measurement	Value	Comments
Signal-to-Noise Ratio (SNR) (fig 2)	226.9	Comparable with BWHM, MINN, STAN and UCLA (ref1)
Signal-to-Fluctuation-Noise ratio (SFNR) (fig 2)	238.1	Comparable with BWHM, MINN, STAN and UCLA (ref1)
Radius of Decorrelation (RDC) (fig 3)	10.8	High value of rdc indicates less scanner instability (ref1)
Drift (fig 1)	0.27	Better than the values reported in ref 1: Indicates stable performance over the time series.
Percent Fluctuation (fig 1)	0.04	Better than the values reported in ref 1: Indicates stable performance over the time series.
Fourier analysis of the residuals (fig 2)	<2	Better than the values reported in ref 1. No abnormally elevated spikes. The system is mechanically stable.
Ghost Mean Percentage (fig 10)	2.15%	Better than the recommended level (3% < ) in ref 2:

Table 1: Key measurements

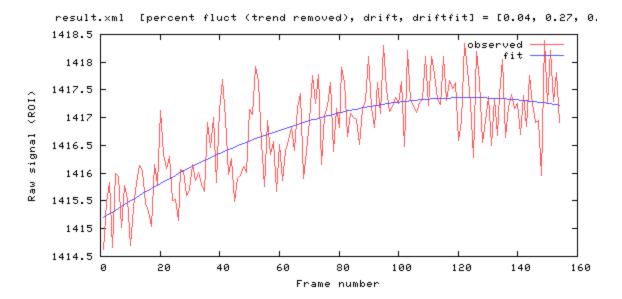


Figure 1

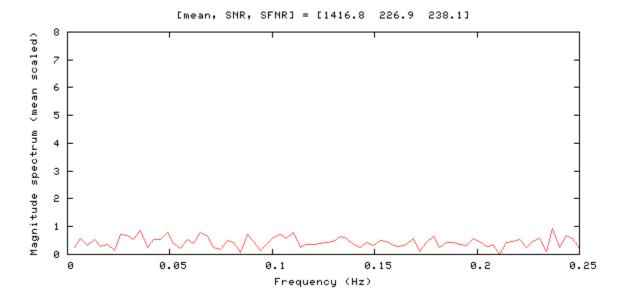


Figure 2

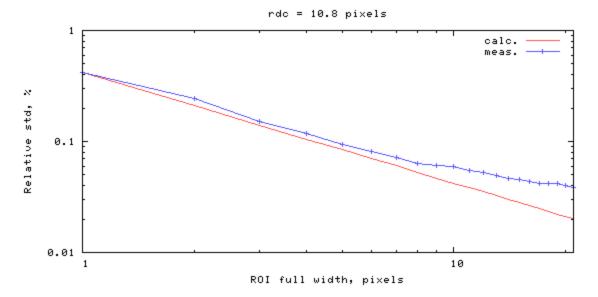


Figure 3

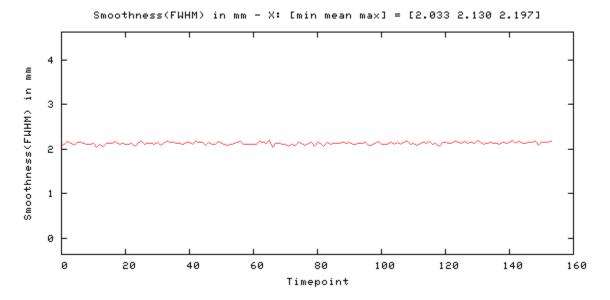


Figure 4

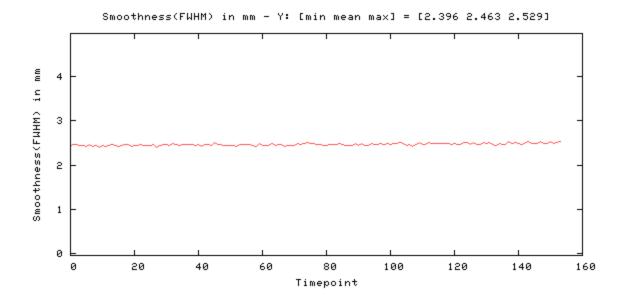


Figure 5

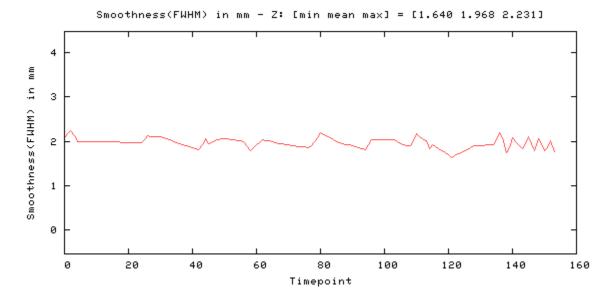


Figure 6

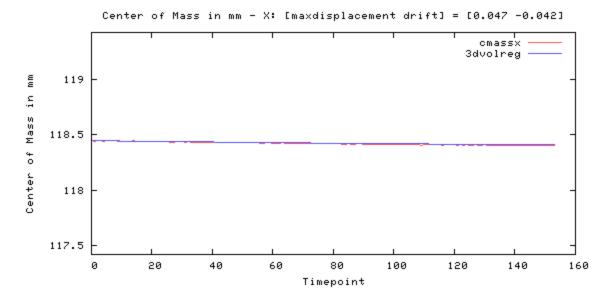


Figure 7

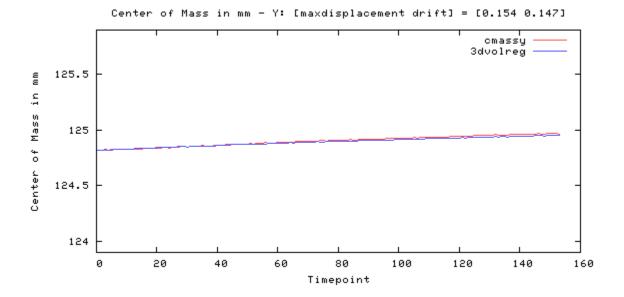


Figure 8

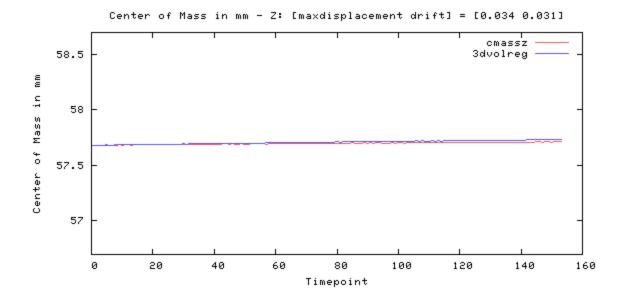


Figure 9

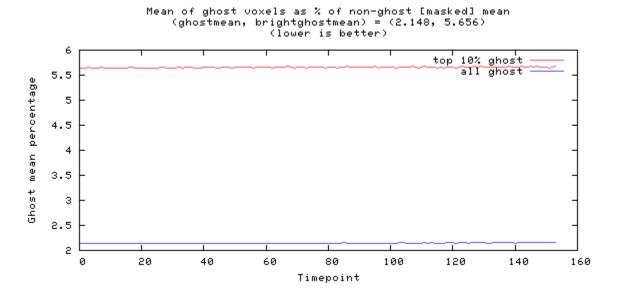


Figure 10

#### Odd-even difference image

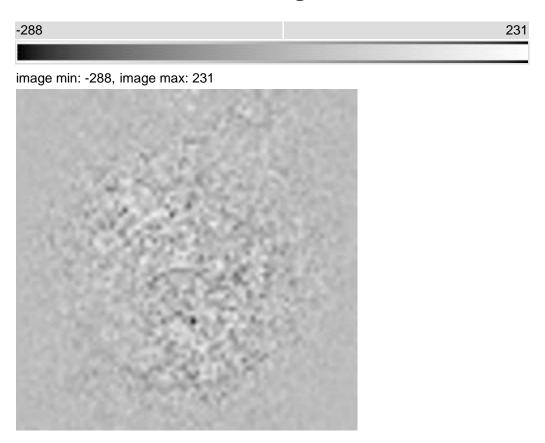


Figure 11

### Mean image

2200.79

image min: 0, image max: 2201.45

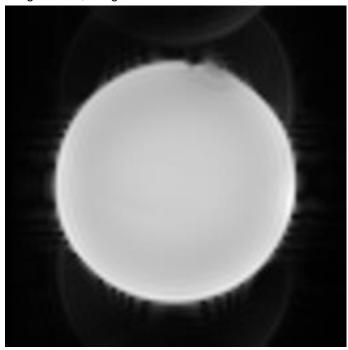


Figure 12

### **Standard Deviation image**

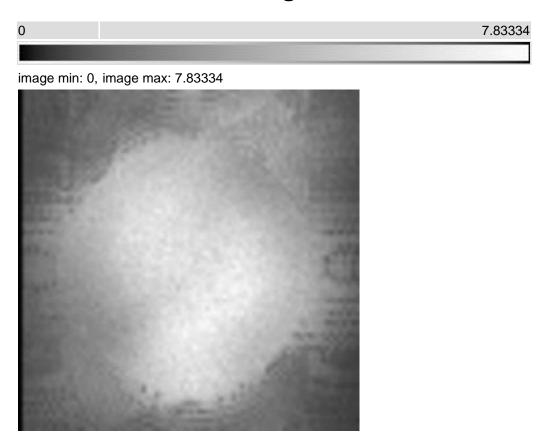


Figure 13

## SFNR image

0 856.734

image min: 0, image max: 910.263

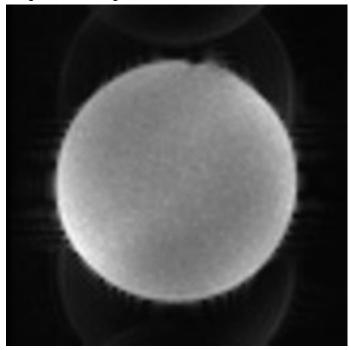


Figure 14

#### **Acquisition parameters**

dimensions	80x80x30x157
spacing	3mmx3mmx3.99mmx2000ms
gap	0mmx0mmx0.49mmx0ms
scanner	AWP66112
psdname	%SiemensSeq%\ep2d_bold
examnumber	22
studyid	QA
seriesnumber	5
runnumber	1
scanningsequence	EP
sequencevariant	SK
magneticfield	3
description	BOLD_p2
scandate	2017-01-17
scantime	09:13:09
tr	2000
te	35.000000
flipangle	90
prescribedslicespacing	3.99
frequencydirection	1
acquisitionmatrix	80 80
bandwidth	185.2
transmitcoilname	Body
institution	Dartmouth College - PBS
examdescription	dbic^QA
scanner	AWP66112
scannermanufacturer	SIEMENS
scannermodelname	Prisma
fieldofview	240 240
psdname	%SiemensSeq%\ep2d_bold
protocolname	BOLD_p2 (BOLD_p2)

institutionaddress	Maynard 3,Hanover,NH,US,03755
institutionaldepartment name	Department
scanoptions	FS
mracquisitiontype	2D
sequencename	*epfid2d1_80
angioflag	N
numaverages	1
imagingfrequency	123.252564
imagednucleus	1H
spacingbetweenslices	3.989999901637
numphaseencodingste ps	79
echotrainlength	39
percentsampling	100
percentphasefieldofvie w	100
scannerserialnumber	66112
softwareversions	syngo MR E11
inplanephaseencoding direction	COL
variableflipangleflag	N
sar	0.05580742565152
dBdt	0
acquisitionnumber	1
instancenumber	1
samplesperpixel	1
photometricinterpretati on	MONOCHROME2
rows	480
columns	480
bitsallocated	16
bitsstored	12
highbit	11
pixelbandwidth	2315
otherinfo	

sliceorder	2,4,6,8,10,12,14,16,18,20,22,24,26,28,30,1,3,5,7,9,11,13,15,1 7,19,21,23,25,27,29
I .	

#### References:

1. Friedman L, Glover GH. Report on a multicenter fMRI quality assurance protocol. J Magn Reson

Imaging. 2006a; 23(6):827-839. [PubMed: 16649196]

- 2. Acceptance Testing and Quality Assurance Procedures for Magnetic Resonance Imaging Facilities, AAPM Report No. 100
- 3. <a href="https://www.nitrc.org/projects/bxh\_xcede\_tools">https://www.nitrc.org/projects/bxh\_xcede\_tools</a>